




SIGVARIS

COMPREFLEX™

INELASTIC COMPRESSION WRAPS



**SIMPLE SOLUTIONS FOR  
LYMPHOEDEMA  
VENOUS INSUFFICIENCY  
WOUND CARE**

# EASILY AND ACCURATELY SET COMPRESSION WITH ACCUTAB

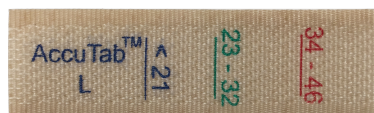


## EASY TO USE

1. Gently lay the COMPREFLEX straps in place without stretching the fabric
2. Attach an ACCUTAB at the end of each strap
3. Pull each strap to the prescribed dosage indicated by the lines on the ACCUTAB:  
<21, 23–32, or 34–46mmHg

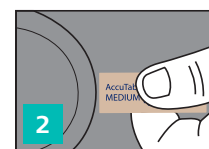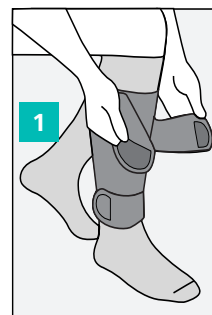

## PROVEN ACCURACY FOR RESTING PRESSURE

### ACCUTAB RESULTS: 23–32mmHg

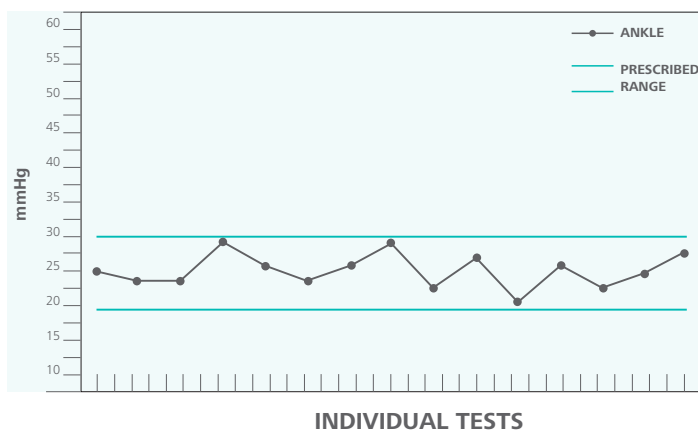

\* ACCUTAB has been rigorously tested according to established guidelines in the International Compression Club Consensus documents 1, 2. Testing was conducted using PicoPress®, which is recognized for its high accuracy and reproducibility 3.

## EFFECTIVE WORKING PRESSURE


- Inelastic compression reduces leg edema and improves venous blood flow 4, 5
- Inelastic compression can increase arterial perfusion in patients with mild to moderate Peripheral Arterial Disease 4, 5

### WORKING PRESSURE

Walking, Stair Climbing, Sit to Stand, Toe Raises

### RESTING PRESSURE

Donning  
COMPREFLEX

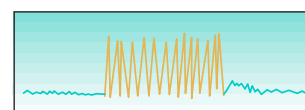

### RESTING PRESSURE

Laying Down  
Sitting Still

## SAFE, PRECISE RESTING PRESSURE

- Inelastic compression can be used safely for patients with mild to moderate Peripheral Arterial Disease 4, 5 \*
- ACCUTAB delivers precision dosages for controlled initial resting pressure when donning SIGVARIS COMPREFLEX garment

# COMPREFLEX LEG WRAP

## HIGH QUALITY

Each garment is double stitched to ensure durability and quality standards

## CONTOURED DESIGN

Garment features a curved design to provide a conforming fit

## STRETCH PANEL

The front stretch panel secures garment in place for easy application and high compliance

## EASY TO USE


Straps roll back and attach to garment to prevent tangling

## COMFORTABLE MATERIALS

Soft, conforming Breathe-O-Prene® provides a comfortable fit and wicks away moisture

## TWO Colour OPTIONS

The COMPREFLEX is available in black or beige

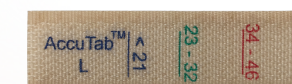

## ACCUTAB

Each COMPREFLEX lower leg wrap includes a set of ACCUTAB compression measuring devices



# COMPREFLEX NO FOOT



## FEATURES

- <21–46mmHg of gradient, short-stretch compression
- Soft, conforming Breathe-O-Prene provides a comfortable fit and wicks away moisture
- Strap extenders may be applied if the patient exceeds standard size ranges (see page 16)
- Good option for patients in need of a different foot option (size or boot)
- Easily worn under most standard footwear Available in black or beige
- Easy care - machine wash and dry
- Latex-free

## INDICATIONS

- Stage I or II Lymphoedema
- Chronic Venous Insufficiency
- Post-surgical Edema

## INCLUDES

- 1 COMPREFLEX BK
- Set of ACCUTAB devices
- Pair of BASIC liners

**5 SIZES AVAILABLE**  
**2 LENGTHS AVAILABLE**  
**2 COLOURS AVAILABLE**

## BASIC LINER Non-compressive




- Easy to apply knee-high liners
- *drirelease* moisture wicking and odor control
- Flat seams for added comfort
- Silver ionized treatment for bacteriostatic properties

## CONTOURED DESIGN

Garment features a curved design to provide a conforming fit

## STRETCH PANEL

The front stretch panel secures garment in place for easy application and high compliance

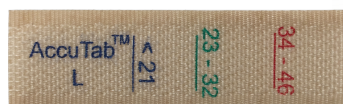

## ACCUTAB

Each COMPREFLEX lower leg wrap includes a set of ACCUTAB compression measuring devices



## EASY TO USE

Straps rolls back and attach to garment to prevent tangling



P.O. #	Account #	Contact	Date		
Phone	Patient Name		Age	Height	Weight

Company Name

Dropship Name & Address


PRODUCT INFORMATION					
Leg	Size	Length	Item #	Color	
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT				<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE	
Leg	Size	Length	Item #	Color	
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT				<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE	

**NOTE:** If foot garment is desired, please see page 14 for options and order form.

## Measuring instructions on page 17

COMPREFLEX - NF SIZING CHART (A-C1)

	SMALL	MEDIUM	LARGE	XLARGE	XXLARGE
C1	29-39cm	34-44cm	40-50cm	46-56cm	54-64cm
C	29-39cm	34-44cm	40-50cm	46-56cm	54-64cm
B	24-34cm	29-39cm	34-44cm	39-49cm	44-55cm
A	19-26cm	21-30cm	26-36cm	31-41cm	36-46cm
BLACK					
REGULAR	1401-NF-BKR	1402-NF-BKR	1403-NF-BKR	1404-NF-BKR	1405-NF-BKR
TALL	1401-NF-BKT	1402-NF-BKT	1403-NF-BKT	1404-NF-BKT	1405-NF-BKT
BEIGE					
REGULAR	1411-NF-BKR	1412-NF-BKR	1413-NF-BKR	1414-NF-BKR	1415-NF-BKR
TALL	1411-NF-BKT	1412-NF-BKT	1413-NF-BKT	1414-NF-BKT	1415-NF-BKT

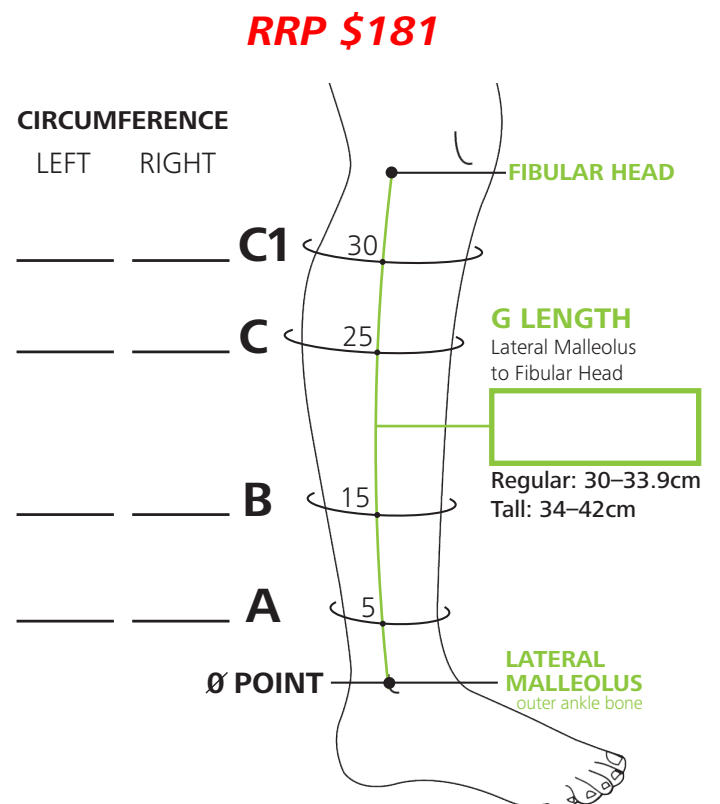

# COMPREFLEX LITE TRANSITION



## FEATURES

- <21–46mmHg of gradient, short-stretch compression
- Soft, conforming Breathe-O-Prene provides a comfortable fit and wicks away moisture
- Strap extenders may be applied if the patient exceeds standard size ranges (see page 16)
- Easily worn under most standard footwear Available in black or beige
- Easy care - machine wash and dry
- Latex-free

## INDICATIONS

- Stage I or II Lymphoedema
- Chronic Venous Insufficiency
- Post-surgical Edema

## INCLUDES

- 1 COMPREFLEX BK
- Set of ACCUTAB devices
- Pair of TRANSITION liners

**5 SIZES AVAILABLE**  
**2 LENGTHS AVAILABLE**  
**2 COLOURS AVAILABLE**

## TRANSITION LINER

10–15mmHg in Foot and Ankle



- Soft, flat seam knee-high liners
- *drirelease* moisture wicking and odor control
- Extra thick sole from toe to heel for added comfort
- Silver ionized treatment for bacteriostatic properties

## CONTOURED DESIGN

Garment features a curved design to provide a conforming fit

## STRETCH PANEL

The front stretch panel secures garment in place for easy application and high compliance



## ACCUTAB

Each COMPREFLEX lower leg wrap includes a set of ACCUTAB compression measuring devices



## EASY TO USE

Straps roll back and attach to garment to prevent tangling

P.O. #	Account #	Contact	Date		
Phone	Patient Name		Age	Height	Weight

Company Name
--------------

Dropship Name & Address

PRODUCT INFORMATION


Leg	Size	Length	Item #	Color
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT				<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE
Leg	Size	Length	Item #	Color
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT				<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE

Measuring instructions on page 17

COMPREFLEX LITE TRANSITION SIZING CHART (A-C1)

	SMALL	MEDIUM	LARGE	XLARGE	XXLARGE
C1	29-39cm	34-44cm	40-50cm	46-56cm	54-64cm
C	29-39cm	34-44cm	40-50cm	46-56cm	54-64cm
B	24-34cm	29-39cm	34-44cm	39-49cm	44-55cm
A	19-26cm	21-30cm	26-36cm	31-41cm	36-46cm
BLACK					
REGULAR	1401-UC-BKR	1402-UC-BKR	1403-UC-BKR	1404-UC-BKR	1405-UC-BKR
TALL	1401-UC-BKT	1402-UC-BKT	1403-UC-BKT	1404-UC-BKT	1405-UC-BKT
BEIGE					
REGULAR	1411-UC-BKR	1412-UC-BKR	1413-UC-BKR	1414-UC-BKR	1415-UC-BKR
TALL	1411-UC-BKT	1412-UC-BKT	1413-UC-BKT	1414-UC-BKT	1415-UC-BKT

RRP \$211

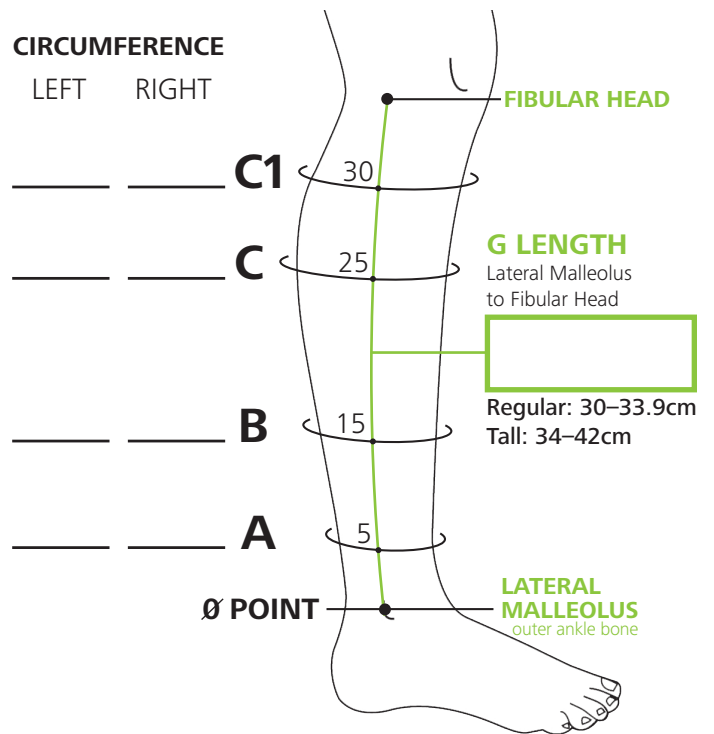

TRANSITION LINER SIZING CHART

SIZE	ANKLE CIRC. (A)	CALF CIRC. (C)	WOMEN'S SHOE	MEN'S SHOE	PAIR (BLACK)	PAIR (BEIGE)
SMALL	19-26cm	29-39cm	<7½	<6½	1601-CK	1611-CK
MEDIUM	21-30cm	34-44cm	7.5-9.5	6.5-8.5	1602-CK	1612-CK
LARGE	26-36cm	40-50cm	10-12	9-11	1603-CK	1613-CK
X-LARGE	31-41cm	46-56cm	12.5-15	11.5-14	1604-CK	1614-CK
XX-LARGE	36-46cm	54-64cm	12.5-15	11.5-14	1605-CK	1615-CK



# COMPREFLEX LITE COMPLETE



## FEATURES

- <21–46mmHg of gradient, short-stretch compression
- Soft, conforming Breathe-O-Prene provides a comfortable fit and wicks away moisture
- Strap extenders may be applied if the patient exceeds standard size ranges (see page 16)
- Easily worn under most standard footwear Available in black or beige
- Easy care - machine wash and dry
- Latex-free

## INDICATIONS

- Stage I or II Lymphoedema
- Chronic Venous Insufficiency
- Post-surgical Edema

## INCLUDES

- 1 COMPREFLEX BK
- Set of ACCUTAB devices
- Pair of COMPLETE liners

**5 SIZES AVAILABLE**  
**2 LENGTHS AVAILABLE**  
**2 COLOURS AVAILABLE**

## COMPLETE LINER

10–15mmHg in Foot, Ankle and Calf



- Comfortable knee-high liners
- *drirelease* moisture wicking and odor control
- Extra thick sole from toe to heel for added comfort
- Silver ionized treatment for bacteriostatic properties

## CONTOURED DESIGN


Garment features a curved design to provide a conforming fit

## EASY TO USE

Straps rolls back and attach to garment to prevent tangling

## STRETCH PANEL

The front stretch panel secures garment in place for easy application and high compliance

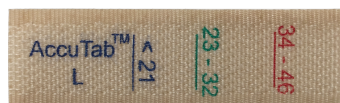

## ACCUTAB

Each COMPREFLEX lower leg wrap includes a set of ACCUTAB compression measuring devices



P.O. #	Account #	Contact	Date		
Phone	Patient Name		Age	Height	Weight

Company Name
--------------

Dropship Name & Address

PRODUCT INFORMATION


Leg	Size	Length	Item #	Color
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT				<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE
Leg	Size	Length	Item #	Color
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT				<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE

Measuring instructions on page 17

COMPREFLEX LITE COMPLETE SIZING CHART (A-C1)

	SMALL	MEDIUM	LARGE	XLARGE	XXLARGE
C1	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
C	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
B	24–34cm	29–39cm	34–44cm	39–49cm	44–55cm
A	19–26cm	21–30cm	26–36cm	31–41cm	36–46cm
BLACK					
REGULAR	1401-FC-BKR	1402-FC-BKR	1403-FC-BKR	1404-FC-BKR	1405-FC-BKR
TALL	1401-FC-BKT	1402-FC-BKT	1403-FC-BKT	1404-FC-BKT	1405-FC-BKT
BEIGE					
REGULAR	1411-FC-BKR	1412-FC-BKR	1413-FC-BKR	1414-FC-BKR	1415-FC-BKR
TALL	1411-FC-BKT	1412-FC-BKT	1413-FC-BKT	1414-FC-BKT	1415-FC-BKT

RRP \$218

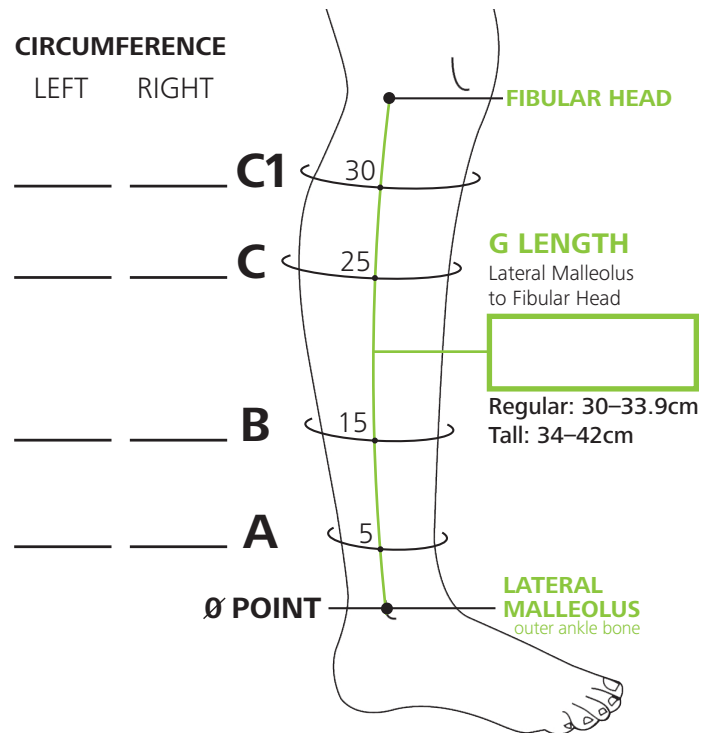

COMPLETE LINER SIZING CHART

SIZE	ANKLE CIRC. (A)	CALF CIRC. (C)	WOMEN'S SHOE	MEN'S SHOE	PAIR (BLACK)	PAIR (BEIGE)
SMALL	19–26cm	29–39cm	<7½	<6½	1601-CK-F1	1611-CK-F1
MEDIUM	21–30cm	34–44cm	7.5–9.5	6.5–8.5	1602-CK-F1	1612-CK-F1
LARGE	26–36cm	40–50cm	10–12	9–11	1603-CK-F1	1613-CK-F1
X-LARGE	31–41cm	46–56cm	12.5–15	11.5–14	1604-CK-F1	1614-CK-F1
XX-LARGE	36–46cm	54–64cm	12.5–15	11.5–14	1605-CK-F1	1615-CK-F1

# COMPREFLEX THIGH COMPONENT



## FEATURES

- Includes ACCUTAB for easy, accurate compression
- Soft, conforming Breathe-O-Prene provides a comfortable fit and wicks away moisture
- Strap extenders may be applied if the patient exceeds standard size ranges (see page 16)
- Combine with COMPREFLEX BK for full leg compression
- 2 KNEE options available
- Available in black or beige
- Easy care - machine wash and dry
- Latex-free

## INDICATIONS

- Stage I or II Lymphoedema
- Chronic Venous Insufficiency
- Post-surgical Edema

## INCLUDES

- 1 COMPREFLEX TC
- Set of ACCUTAB devices
- Belt & Hip Attachment
- Pair of non-compressive cotton liners

**4 SIZES AVAILABLE**  
**2 LENGTHS AVAILABLE**      **L + R**  
**2 COLOURS AVAILABLE**



### BELT & HIP ATTACHMENT

These items are included to ensure a comfortable fit, and help secure garment in place

### EASY TO USE

Straps rolls back and attach to garment to prevent tangling

### STRETCH PANEL

The front stretch panel secures garment in place for easy application and high compliance

## KNEE OPTIONS *(additional charge)*



### COMPREFLEX KNEE

#### Closed Patella

Soft, conforming compression with straps that wrap around the back of the knee



### COMPREFLEX KNEE

#### Open Patella

Soft, conforming compression with straps that wrap around the back of the knee

**NOTE:** Please see page 16 for options.

**3 SIZES AVAILABLE**  
**2 COLOURS AVAILABLE**



P.O. #	Account #	Contact	Date
Phone	Patient Name	Age	Height Weight

Company Name

Dropship Name & Address

## PRODUCT INFORMATION

Leg	Size/Length	Item #	Color
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT			<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE
Leg	Size/Length	Item #	Color
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT			<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE
KNEE STYLE	Item #	Size	Color
<input type="checkbox"/> OPEN PATELLA <input type="checkbox"/> CLOSED PATELLA			<input type="checkbox"/> BLACK <input type="checkbox"/> BEIGE

## Measuring instructions on page 17

### COMPREFLEX - TC SIZING CHART (D-F) **RRP \$213**

	SMALL	MEDIUM	LARGE	XLARGE
F	48-58cm	56-66cm	64-74cm	74-84cm
E	43-53cm	51-61cm	58-68cm	68-78cm
D	38-48cm	46-56cm	53-63cm	63-73cm

LEFT

BLACK	REGULAR	1401-TCR-L	1402-TCR-L	1403-TCR-L	1404-TCR-L
	TALL	1401-TCT-L	1402-TCT-L	1403-TCT-L	1404-TCT-L

RIGHT

BLACK	REGULAR	1401-TCR-R	1402-TCR-R	1403-TCR-R	1404-TCR-R
	TALL	1401-TCT-R	1402-TCT-R	1403-TCT-R	1404-TCT-R

LEFT

BEIGE	REGULAR	1411-TCR-L	1412-TCR-L	1413-TCR-L	1414-TCR-L
	TALL	1411-TCT-L	1412-TCT-L	1413-TCT-L	1414-TCT-L

RIGHT

BEIGE	REGULAR	1411-TCR-R	1412-TCR-R	1413-TCR-R	1414-TCR-R
	TALL	1411-TCT-R	1412-TCT-R	1413-TCT-R	1414-TCT-R

### COMPREFLEX - TC SIZING CHART (D-F) **RRP \$85**


	SMALL	MEDIUM	LRG / X-LRG
D	38-48	46-56	53-73
C1	29-39	34-44	39-65

CLOSED PATELLA

BLACK	1101-KP	1102-KP	1103-KP
BEIGE	1111-KP	1112-KP	1113-KP

OPEN PATELLA

BLACK	1101-KP-OP	1102-KP-OP	1103-KP-OP
BEIGE	1111-KP-OP	1112-KP-OP	1113-KP-OP

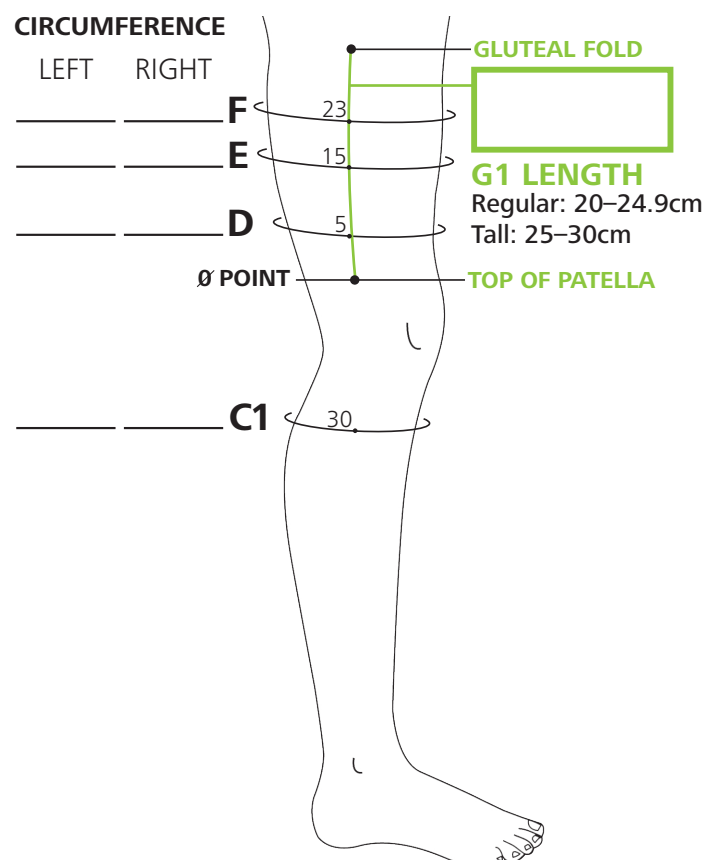

# FOOT COMPRESSION

## COMPRESSBOOT LITE



**2 SIZES AVAILABLE**  
**2 LENGTHS AVAILABLE**  
**COMPRESSBOOT**

10–15mmHg

- Ideal for mild edema/lymphoedema
- Ultra-thin profile that easily fits into most footwear
- Highly adjustable design (may be trimmed to fit)
- Breathable stretch fabric conforms to ankle
- Provides mild, therapeutic compression

COMPRESSBOOT LITE PRODUCT CODES		
	SMALL/ MEDIUM / LARGE	XLRG / XX LRG
REGULAR	1803-BLR	1805-BLR
LONG	1803-BLL	1805-BLL

**RRP \$32**

30–40mmHg

- Ideal for mild-moderate edema/lymphoedema
- 4-way stretch for firm compression around ankle
- Features neoprene material with hook and loop tabs
- Provides moderate, therapeutic compression

COMPRESSBOOT PRODUCT CODES			
	SMALL	MEDIUM/LARGE	XLRG / XX LRG
BLACK			
REGULAR	1801-BTR	1803-BTR	1805-BTR
LONG	1801-BTL	1803-BTL	1805-BTL
BEIGE			
REGULAR	1811-BTR	1813-BTR	1815-BTR
LONG	1811-BTL	1813-BTL	1815-BTL

**RRP \$87**

30–40mmHg

- Ideal for chronic/acute edema/lymphoedema
- Chipped foam lining
- Removable gripper patches provide traction
- Straps are removable for custom positioning or replacement
- Provides moderate, therapeutic compression

COMPRESSBOOT PLUS PRODUCT CODES			
	SMALL	MEDIUM / LARGE	XLRG / XX LRG
REGULAR	1801-BPR	1803-BPR	1805-BPR
LONG	1801-BPL	1803-BPL	1805-BPL

**RRP \$331**

## COMPRESSBOOT PLUS



**3 SIZES AVAILABLE**  
**2 LENGTHS AVAILABLE**


P.O. #	Account #	Contact	Date		
Phone	Patient Name		Age	Height	Weight
Company Name					
Dropship Name & Address					

PRODUCT INFORMATION	
Foot	
<input type="checkbox"/> LEFT <input type="checkbox"/> RIGHT	
Size	<input type="checkbox"/> COMPREBOOT LITE
Length	<input type="checkbox"/> COMPREBOOT
Item #	<input type="checkbox"/> COMPREBOOT PLUS

## MEASURING INSTRUCTIONS

- I:** Measure circumference of ankle bend and heel.
- J:** Measure circumference across metatarsal heads.
- K:** Measure length on the medial side of the foot from the heel to the 1st metatarsal head.

CIRCUMFERENCE	
LEFT	RIGHT
_____	_____ <b>I</b>
_____	_____ <b>J</b>
_____	_____ <b>K</b>

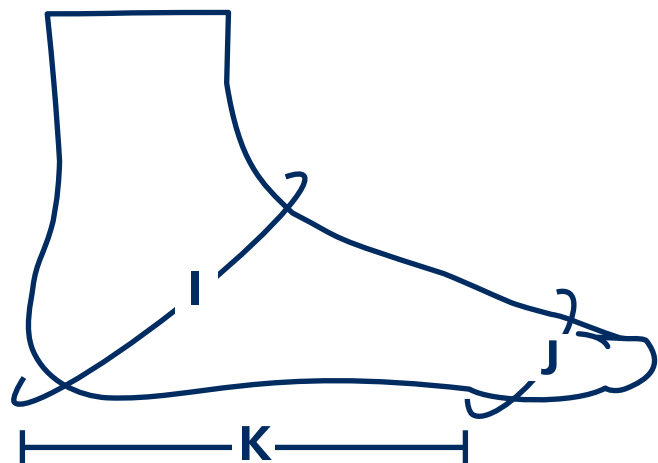

FOOT SIZING CHART (I-K)

	SMALL		MED / LARGE		XL / XXL	
	REGULAR	LONG	REGULAR	LONG	REGULAR	LONG
I	28-36cm	28-36cm	≤39cm	≤39cm	≤44cm	≤44cm
J	22-26cm	22-26cm	≤30cm	≤30cm	≤33cm	≤33cm
K	14-18cm	19-23cm	18-20cm	23-25cm	20-22cm	25-27cm



# ACCESSORIES & LINERS



## COMPREFLEX STRAP EXTENDER

Adds 10 cm of adjustable range to a single strap.

1 SET

BLACK	1401-SE1	<b>RRP \$10</b>
BEIGE	1411-SE1	



## COMPREFLEX

Compression garment with either open or closed patella (ACCUTAB not required).

	SMALL	MEDIUM	LRG / X-LRG
D	38-48	46-56	53-73
C1	29-39	34-44	39-65
CLOSED PATELLA			
BLACK	1101-KP	1102-KP	1103-KP
BEIGE	1111-KP	1112-KP	1113-KP

OPEN PATELLA

BLACK	1101-KP-OP	1102-KP-OP	1103-KP-OP
BEIGE	1111-KP-OP	1112-KP-OP	1113-KP-OP

**RRP \$85**



## BASIC LINER

Non-compressive


- Easy to apply knee-high liners
- *drirelease* moisture wicking and odor control
- Flat seams for added comfort
- Silver ionized treatment for bacteriostatic properties

Size: One size fits all

PAIR

1700-NK1

**RRP \$44**

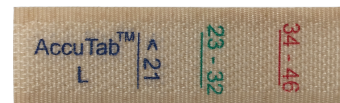

**RRP \$15**

## ACCUTAB

Compression measuring device design to use with COMPREFLEX leg wraps.

SIZE	1 SET OF 5
SMALL	1491-TB1
MEDIUM	1492-TB1
LARGE	1493-TB1
X-LARGE	1494-TB1
XX-LARGE	1495-TB1



## COMPLETE LINER

10-15mmHg in Foot, Ankle & Calf

- Comfortable knee-high liners
- *drirelease* moisture wicking and odor control
- Extra thick sole from toe to heel for added comfort
- Silver ionized treatment for bacteriostatic properties

**RRP \$63**



## TRANSITION LINER

10-15mmHg in Foot & Ankle

- Soft, flat seam knee-high liners
- *drirelease* moisture wicking and odor control
- Extra thick sole from toe to heel for added comfort
- Silver ionized treatment for bacteriostatic properties

**RRP \$54**

### COMPLETE LINER SIZING CHART

SIZE	ANKLE CIRC. (A)	CALF CIRC. (C)	WOMEN'S SHOE	MEN'S SHOE	PAIR (BLACK)	PAIR (BEIGE)
SMALL	19-26cm	29-39cm	<7½	<6½	1601-CK-F1	1611-CK-F1
MEDIUM	21-30cm	34-44cm	7.5-9.5	6.5-8.5	1602-CK-F1	1612-CK-F1
LARGE	26-36cm	40-50cm	10-12	9-11	1603-CK-F1	1613-CK-F1
X-LARGE	31-41cm	46-56cm	12.5-15	11.5-14	1604-CK-F1	1614-CK-F1
XX-LARGE	36-46cm	54-64cm	12.5-15	11.5-14	1605-CK-F1	1615-CK-F1

### TRANSITION LINER SIZING CHART

SIZE	ANKLE CIRC. (A)	CALF CIRC. (C)	WOMEN'S SHOE	MEN'S SHOE	PAIR (BLACK)	PAIR (BEIGE)
SMALL	19-26cm	29-39cm	<7½	<6½	1601-CK	1611-CK
MEDIUM	21-30cm	34-44cm	7.5-9.5	6.5-8.5	1602-CK	1612-CK
LARGE	26-36cm	40-50cm	10-12	9-11	1603-CK	1613-CK
X-LARGE	31-41cm	46-56cm	12.5-15	11.5-14	1604-CK	1614-CK
XX-LARGE	36-46cm	54-64cm	12.5-15	11.5-14	1605-CK	1615-CK

*Follow contour of the arm/leg by placing the measuring tape against the skin. Measure in centimeters.*

## BELOW KNEE

---

### **CIRCUMFERENCES**

Measure circumferences where indicated by the black dots, beginning with the dot 5cm above the **Lateral Malleolus** (outer ankle bone), continue to the 15cm, 25cm and 30cm dots. Record in the corresponding lines A, B, C, C1(Left/Right).

### **LENGTH**

Measure length from the **Lateral Malleolus** (outer ankle bone) to lateral aspect of the **Fibular Head**, and record in box G.

### **FOOT**

Measure circumference of ankle bend and heel and record on line I. Measure circumference across metatarsal heads and record on line J. Measure length on medial side of foot from heel to 1st metatarsal head and record on line K.

## THIGH COMPONENT

---

### **CIRCUMFERENCES**

Measure circumferences where indicated by the black dots, beginning with the dot 5cm above the lateral aspect **Top of Patella** and continue to the 15cm, and 23cm dots. Record in the corresponding lines D, E, F (Left/Right).

### **LENGTH** (COMPREFLEX only)

Measure length from the lateral aspect **Top of Patella** to lateral aspect **Gluteal Fold**, and record in box G1.

## THIGH HIGH

---

### **CIRCUMFERENCES**

Measure circumferences where indicated by the black dots, beginning with the dot 5cm above the **Lateral Malleolus** (outer ankle bone) and continue to the 15cm, 25cm and 30cm dots. Record in the corresponding lines A, B, C, C1(Left/Right). Measure circumferences at the 5cm dot from the lateral aspect **Top of Patella** and continue to the 15cm, and 23cm dots. Record in the corresponding lines D, E, F (Left/Right).

### **LENGTH**

Measure length from the **Lateral Malleolus** (outer ankle bone) to lateral aspect of the **Gluteal Fold**, and record in box H.

### **FOOT**

Measure circumference of ankle bend and heel and record on line I. Measure circumference across metatarsal heads and record on line J. Measure length on medial side of foot from heel to 1st metatarsal head and record on line K.

*For more information, please speak with your  
SIGVARIS Account Manager  
Michelle Palmieri: 0403 343 129  
or  
Customer Care: 8332 1666.*

SIGVARIS

# COMPREFLEX™

INELASTIC COMPRESSION WRAPS



Phone

**08 8332 1666**

Email

**[sales@surmed.com.au](mailto:sales@surmed.com.au)**

**[surmed.com.au](http://surmed.com.au)**

\* Safe compression dosage should be prescribed by a physician based on individual patient medical diagnoses and needs. Compression should not be applied on patients with severe Peripheral Arterial Disease as tissue necrosis, limb loss, or death may result. When using ACCUTAB, do not apply more mmHg of resting compression pressure than prescribed by your physician. Immediately remove your compression garment and consult with your physician if you experience leg pain, numbness or tingling, coolness, or any other adverse symptoms.

(1) Partsch, Hugo, et al. "Measurement of lower leg compression in vivo: recommendations for the performance of measurements of interface pressure and stiffness." *Dermatologic Surgery* 32.2 (2006): 224–233.

(2) <http://www.tagungsmanagement.org/comp/index.php/icc-consensus>

(3) Mosti, Giovanni, and Susanna Rossari. "The importance of measuring sub bandage pressure and presentation of new measuring device." *Acta Vulnol* 6 (2008): 31–36.

(4) Mosti, Giovanni, Maria Letizia Iabichella, and Hugo Partsch. "Compression therapy in mixed ulcers increases venous output and arterial perfusion." *Journal of vascular surgery* 55.1 (2012): 122–128.

(5) Mosti, Giovanni. "Compression in mixed ulcers: venous side." *Phlebology* 29.1\_suppl (2014): 13–17.

SIGVARIS, COMPREFLEX, Breathe-O-Prene, and ACCUTAB are registered trademarks of SIGVARIS AG, CH-9014 St.Gallen/Switzerland, in many countries worldwide. © Copyright 2017 SIGVARIS, Inc. 2017 *drrelease*. *drrelease* is an Optimier Brands company. All Rights Reserved.

LIT-COMPREFLEXBOOK 09.17

Disclaimer: The information contained on this brochure is for general information purposes only. Surgical and Medical Pty Ltd accepts no responsibility for errors or omissions in the contents. For further clarification on any aspect of the information contained herein, please refer to the technical data sheets from the manufacturer.