



SURGICAL AND MEDICAL SUPPLIES PTY. LTD. ABN 51 007 707 128

16 Kensington Road Rose Park South Australia 5067

PO Box 2 Kent Town SA 5071

Telephone: (08) 8332 1666 Facsimile: (08) 8332 5132

Home Page: <http://www.surmed.com.au>

Email: sales@surmed.com.au



ADHESION INSTEAD OF SUTURES!



EPIGLU®

THE INNOVATIVE TISSUE ADHESIVE
FOR EFFECTIVE WOUND TREATMENT



EPIGLU® - THE ETHYL-2-CYANOACRYLATE ADHESIVE FOR WOUND TREATMENT OF THE HIGHEST STANDARD

EPIGLU®

is the Ethyl-2-Cyanoacrylate adhesive for the closure of wounds on the skin surface. Irrespective of the length of the wound (also surface wounds following removal of skin lesions), all cuts, lacerations and surgical wounds can be treated, that no longer bleed and are not older than 6 hours. **EPIGLU®** can also be used on the oral mucosa and the external genitalia. **EPIGLU®** polymerizes within seconds and adheres even long wounds which are under tension.

Adhesion instead of sutures: Economical, effective, time-saving

Advantages for the patient:

- Wounds can be treated completely painlessly with **EPIGLU®**
- No tension pain as with sutures and no removal of stitches
- Faster healing process
- More aesthetic scarring
- Immediate showering / washing possible

Advantages for the user:

- Economical
- Versatile applications
- Adheres even in difficult places
- Can also be used on wounds that are under tension
- Polymerizes very quickly
- Long shelf-life
- Protects the wound from infections
- Easy to use
- Hardly any keloid formation
- No allergic reactions known
- No further visit to the doctor necessary, as **EPIGLU®** detaches itself after successful reepithelialization

In the reasonably priced tube



or innovative Single Dose



CASE STUDIES


Treatment of a laceration on the lip



When the wound has stopped bleeding the wound edges are fixed in position at one point and then completely covered with **EPIGLU®**. After 11 days the injury is barely noticeable.



Treatment of Aphthae

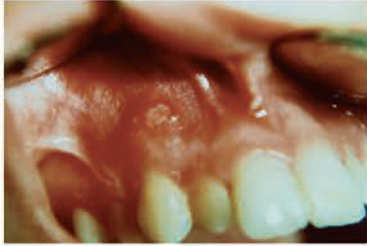

After applying **EPIGLU®** to the painful aphtha the patient is immediately free of pain. Normally the aphtha has healed after 4 days.

Attaching of skin substitute material



The attaching of a skin substitute material to the removal point of the split skin is less expensive and saves time compared to traditional methods. The healing process is fast, and above all, infection free.

Closure of the thorax after bypass surgery



With this long surgical wound, the epidermis was sealed with two layers of **EPIGLU®** after subcutaneous sutures. Wound healing proceeded as in all other examples, without any problems or keloid formation.

EXPERTS OPINION

Dr. Jürgen Reus MD, Plastic and Aesthetic Surgery, Karlsruhe, Germany: „The application of **EPIGLU®** is fast, simple and economical. If used properly no heat is generated. Even wounds that are under slight tension can be adhered. “

Prof. Dr. Rolf R. Lehmann, University of Muenster, Germany: „The results with **EPIGLU®** lead to a significantly better wound healing than by the conventional suturing of the wound ... Furthermore, the application is painless. “

Prof. Dr. Günther Heinrich Willital, Pediatric Surgery, Muenster, Germany: „Wound adhesion with **EPIGLU®** results in an optimal wound closure for infants and children . The application is not only simple, effective and economical, but also leads to a cosmetically flawless result. “

Dr. Shim Aesthetic Surgery, Seoul, Korea: „With **EPIGLU®** I achieve better results than I ever achieved with any other wound stabilizer. I am just fascinated by the results and use it now in every operation.... The quota of hypertrophic scar formation is reduced tremendously when bonding with **EPIGLU®** . “

Dr. Hans H. Sellmann DDS, Marl; Germany:
„**EPIGLU®**, the wound treatment for many cases, as there is always something to glue. “





SURGICAL AND MEDICAL SUPPLIES PTY. LTD. ABN 51 007 707 128

16 Kensington Road Rose Park South Australia 5067
PO Box 2 Kent Town SA 5071
Telephone: (08) 8332 1666 Facsimile: (08) 8332 5132
Home Page: <http://www.surmed.com.au>
Email: sales@surmed.com.au



EPIGLU® - AFFORDABLE - PRACTICAL - EFFECTIVE



For the doctor's practices, emergency room and the care sector

Product	Description	Contents	Order number
EPIGLU® Small package	for 20 applications	1 EPIGLU®-Tube 3 g with 20 application sets	EPIGLU01



For the single use in doctor's practices, emergency room and the care sector

Product	Description	Contents	Order number
EPIGLU® Single Dose 10 pcs. package	for 10 applications	10 Aluminium pouches each 0.3 ml-Dosette EPIGLU®	EPIGLUS12
EPIGLU® Single Dose 5 pc. package	for 5 applications	5 Aluminium pouches each 0.5 ml-Dosette EPIGLU®	EPIGLUS18

EPIGLU®

**THE SOLUTION FOR EFFECTIVE WOUND TREATMENT IN PRACTICES,
EMERGENCY ROOM AND THE OPERATION THEATRE**



Meyer-Haake GmbH
Medical Innovations
Daimlerstrasse 4
61239 Ober-Moerlen
Germany

☎ +49 (0) 6002-99270-0
☎ +49 (0) 6002-99270-22
✉ export@meyer-haake.com
🌐 www.meyer-haake.com

Distributor:



SURGICAL AND MEDICAL SUPPLIES PTY LTD
16 Kensington Road Rose Park SA 5067
Telephone: (08) 8332 1666 Fax: (08) 8332 5132
Email: sales@surmed.com.au
Website: www.surmed.com.au